



Measuring Health

Straightforward – Accurate – Revealing
The Heart Rate Variability Analysis of Nilas MV®



Devices for holistic Health and Medicine



VitaCellMedical GmbH

Am Sonnenhang 44
D-67714 Waldfischbach-Burgalben
Tel. +49 6333 9909030
E-Mail info@vitacellmedical.com
Internet www.vitacellmedical.com


Content Overview

Heart Rate Variability	3
Nilas MV®: Unsurpassed value to your work	4
High level quality standards meet operative efficiency	5
Detecting the root cause of symptoms	6
Understanding health issues und measures	6
Facilitate the communication with your patients	6
Visualise the efficacy of your treatment!	7
HRV monitoring for long-term measurements	8
Complex biological processes made fully transparent	9
Versatile functions for multi-purpose needs	10
Right on the heartbeat - The Rhythmogram	10
The dynamics of health	10
The autonomic nervous system - unconscious nervous system	11
Central Regulation	12
Psycho-emotional state	13
The harmony of biorhythms	13
The biological age	13
Everything at a glance: The overall assessment	14
Progress monitoring	14
The Chart of the Meridians	15
Bio-Energetic Field	15
Chakras Activity Map	15
Ayurveda Test: The Doshas and the 5 Elements	16
Overview of Parameters	16
The power of breath for health and fitness -	
The Breathing Module of Nilas MV®	17
Getting back to balance by setting the right tone -	
The Sound Module of Nilas MV®	17
Scope of Delivery	18
Space for your notes	19

Heart Rate Variability

The smart health index

Did you know that our heart is a resonating body for many vitality processes and biorhythms steered by the central and autonomous nervous system? It is inter alia closely linked with our brain via neurotransmitters as well as with the organ, immune and endocrine system and perceives every little change occurring there. The activity level of these vital functions can be extracted from the heart rate variability and qualitatively evaluated. Hence, HRV is a global indicator for identifying the individual health status.

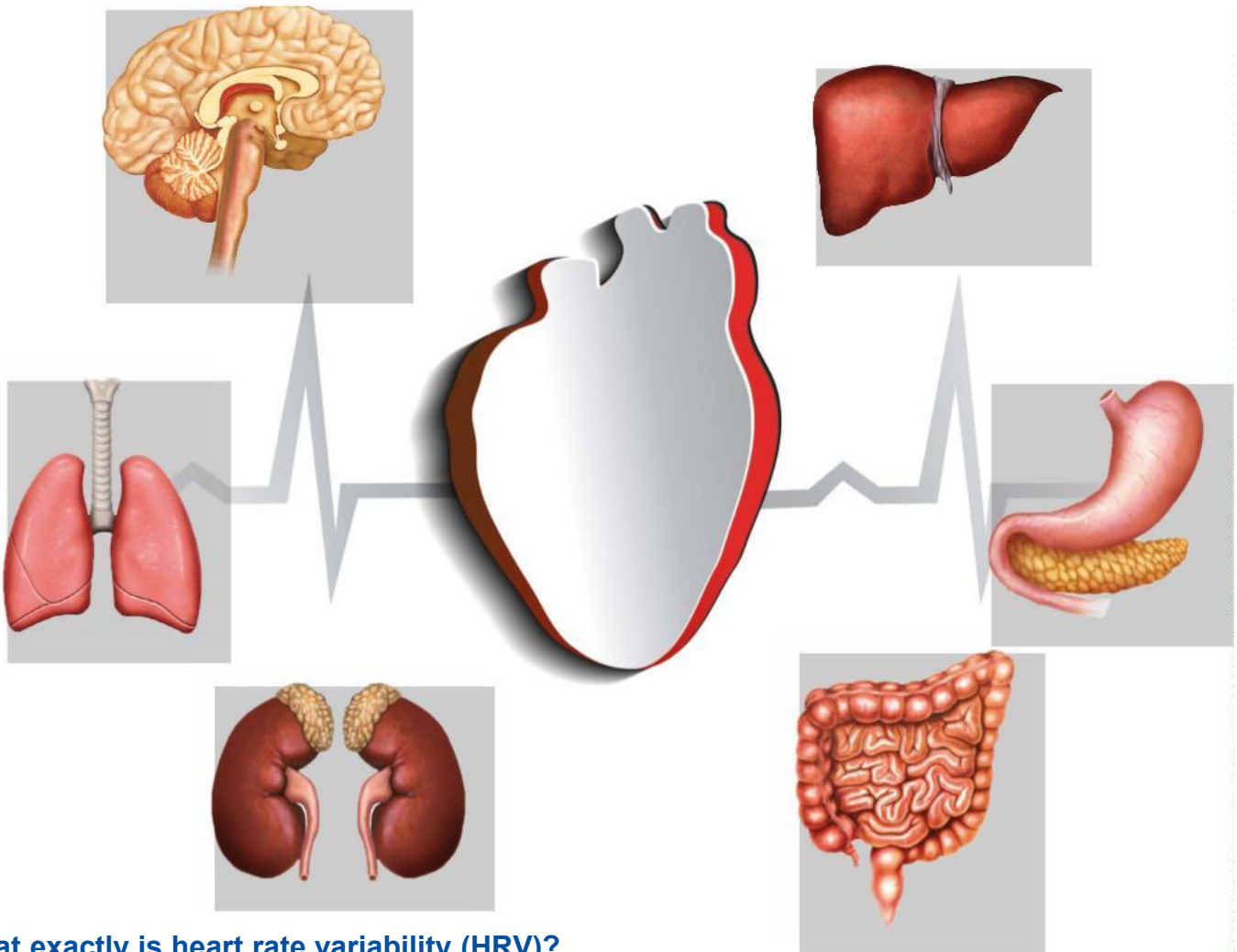


What exactly is heart rate variability (HRV)?

Heart Rate Variability (HRV) refers to the heart's regulatory capacity to vary the time interval between heartbeats consecutively in response to a variety of sources of stress-induced stimuli. It is a reliable performance indicator of the organism delivering sound information of the heart's and the body's overall health. The concept of HRV, when it is within normal parameters, is considered to be a welcome phenomenon.

How does Nilas MV® extract the HRV?

With a measurement of the cardiac cycle Nilas identifies how variable the heart operates to compensate and regulate stress at an optimum level. For this purpose mean values and standard deviations of so-called RR-intervals are used.

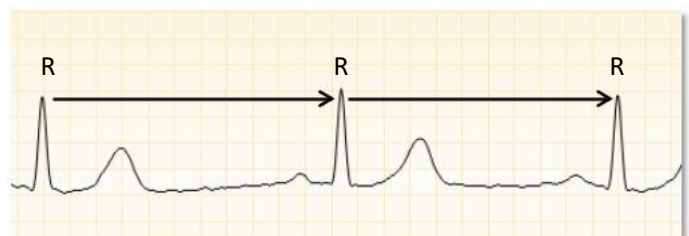

Heart beats illustrated as RR-intervals

Nilas MV®: Unsurpassed value to your work

Medical Application

- Cardiovascular and metabolic risk determination
- Diagnostic work up of acute cases
- Post-infarction check up
- Basic clinical diagnostic method of diabetic neuropathy
- Physical and psychological therapy and rehabilitation monitoring
- Progress and success evaluation of therapeutic measures

Prevention

- Stress analysis and burn-out prevention
- Risk evaluation, severity of stress, therapy response (e. g. with chronic stress and burn-out)
- Gerontological regulatory capacity
- Sleep-wake cycle timeline

Nutrition counselling

- Metabolic functioning and regulation
- Acid-base balance
- Body weight management
- Digestion and obesity issues
- Nutritional changes

Popular and professional sports

- Performance diagnostics and intensity regulation in the training division
- Monitoring parameters for (over)training
- Regeneration monitoring and optimisation

Coaching

- Situational and personality analysis
- Holistic interpretation
- Assessment of personal competencies
- Awareness raising on self-reflection
- Managing resources
- Strategy of communication
- Overcoming conflicts
- Psycho-educative personal management training

For . . .

- Physicians
- Natural Health Professionals
- Manual therapists
- Pharmacists
- Dietician
- Corporate healthcare management
- Coaches and trainers

. . . for the healthiness and well-being of your patients and clients

High level quality standards meet operative efficiency


Convenient and practical application

The HRV-Scan is conducted with the aid of two non-invasive clip electrodes, suitably for wrists and foot joints - **without undressing**. The user-friendly measuring appliance therefore permits a direct integration into the daily work routine and can be easily delegated to assistants.



Reliable measuring technique - Genuine heartbeat recording

The measuring application of Nilas MV® serves to record human biological signals as a solid basis for diagnostics of the organism's functional state. In contrast to a pulse measuring procedure which only records a pulse wave without a specific punctum maximum (point of maximal impulse) the Nilas MV® cardiograph records the exact R wave peaks.

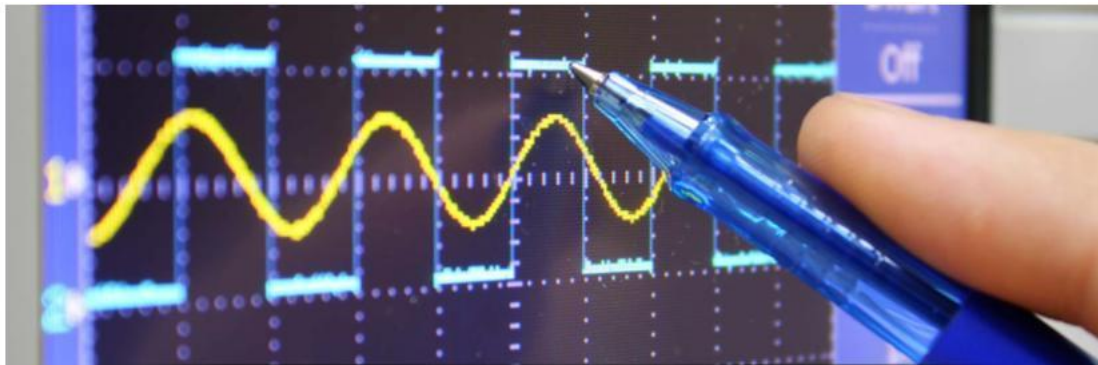

In this way Nilas MV® provides measuring results in an accuracy margin of one millisecond with the highest precision, capturing the exact length and intensity of every single heartbeat. This measuring accuracy is verifiable in a metrological test during which randomly fed-in amplitudes are recorded without any deviation by Nilas MV®.

Information security and free of electromagnetic smog

Nilas MV® works completely without any radio signals such as WLAN or Bluetooth, which are used with some HRV systems. Those procedures impact the data consistency as well as the biorhythms of the organism. Thus they distort the picture about the actual health status of the person tested.



Detecting the root cause of symptoms

Ahead of every health measure a thorough analysis and evaluation of the current situation is required. Only in this way a tailor-made and effective aid can be developed and provided.

Nilas MV® offers a variety of unique parameter functions which illustrate objectively the stress situation of the organism that can be a trigger of discomfort as well as symptom of health impairments.

Understanding health issues und measures


According to a representative study conducted by the University of Bielefeld 54.3 % of patients in Germany are unable to cope with health-related information. They have difficulties in comprehending relevant content presented by their therapist, in assessing treatment options suggested or in being able to relate to the effects of applied measures.

Hence, it is crucial for even the most technically qualified therapist to be able to successfully enlighten the patient on his own situation and in this way increase his compliance to become actively involved in treatment measures.



Facilitate the communication with your patients


For this purpose Nilas MV® pays particular attention to the explanatory power and comprehensibility of the biological data gathered during the HRV measurement. The compiled information is appealingly depicted in a colour-coded and/or numerical assessment system. This allows for an easy entry into the work with Nilas MV® and a time-efficient evaluation routine in your everyday practice. At the same time the informative overviews also help your patients to better understand their overall situation.

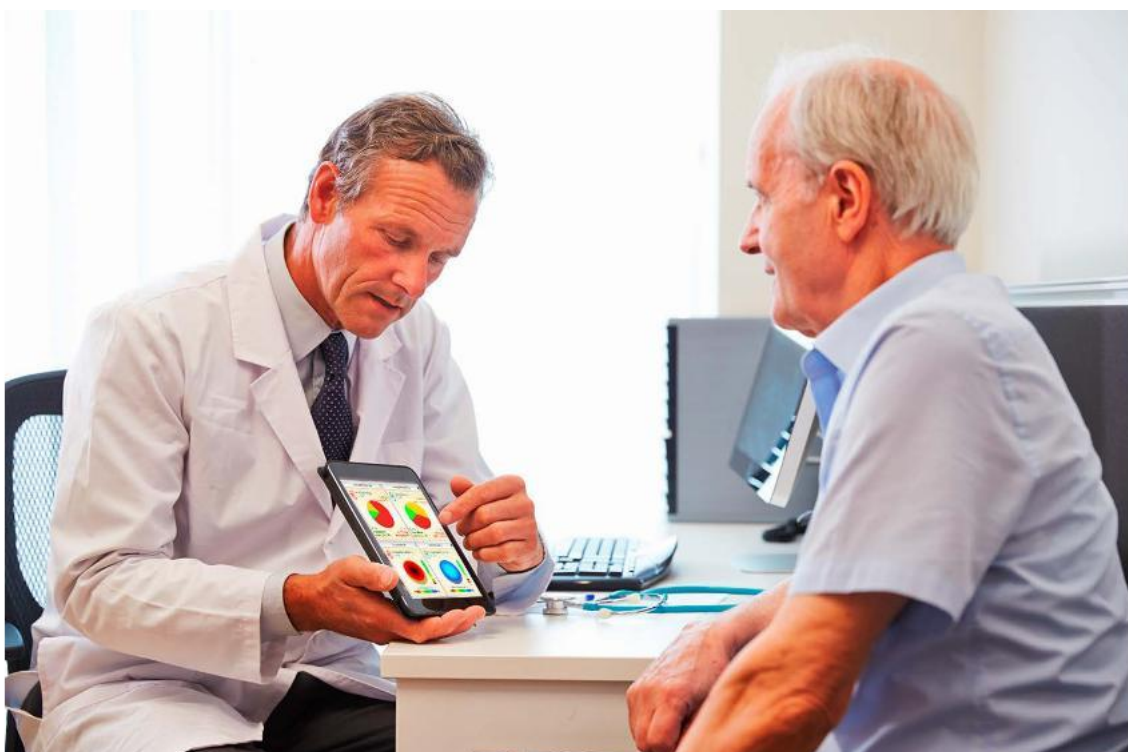


Visualise the efficacy of your treatment!

Enhance your patients' compliance to stick to treatment plans by providing evidence on improvements already at a stage when benefits are not yet subjectively perceptible.


In just about 5 minutes Nilas MV® provides a comprehensive assessment of biorhythmic health parameters, which deliver information about the development of systemic core values and the regulatory capacity of a patient.

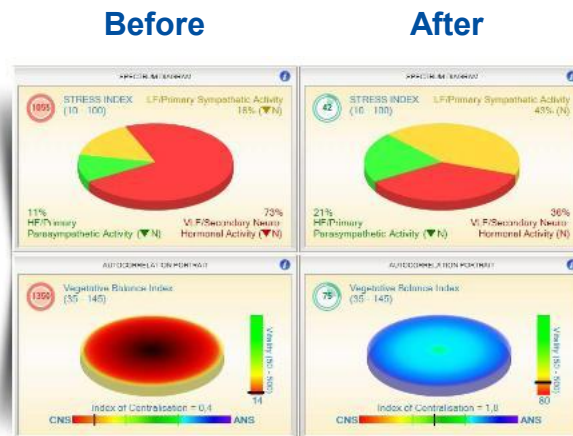

The measurement of heart rate variability with Nilas MV® displays explanatory and understandably alterations of physiological, neuro-endocrine and psycho-emotional origin.

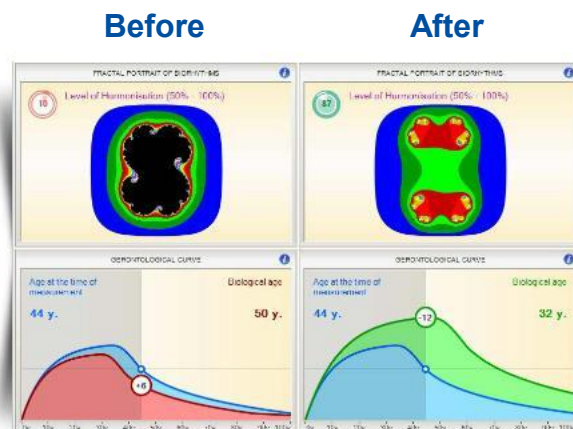


A major asset for you: The software allows comparative measurements to support effectively and efficiently your success and progress monitoring with therapeutic applications.

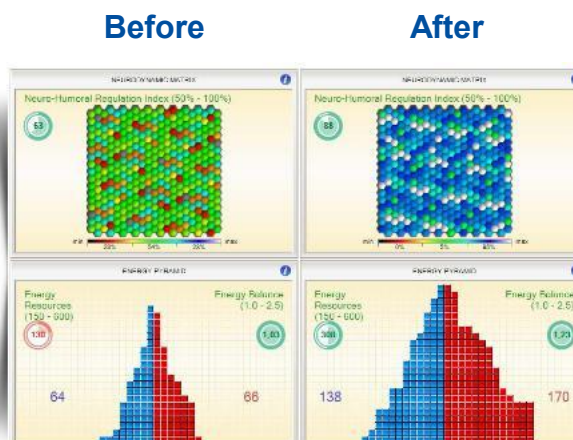


Nilas MV® offers an integrated report function to export and document the measuring results. With the separately available add-on module "Vitality Report" a personal status report for patients/clients can be created that helps to further facilitate the understanding of the results' context.

HRV monitoring for long-term measurements

The integrated "Monitoring" application allows you to carry out self-defined long-term measurements or measurements beyond 300 heartbeats. It is suitable e. g. to evaluate the efficacy of therapeutic interventions and compounds on a real-time basis e. g. medication, infusions, food supplements, bioidentical hormones, bio-resonance, oxygen therapy, acupuncture, magnetic field therapy etc.

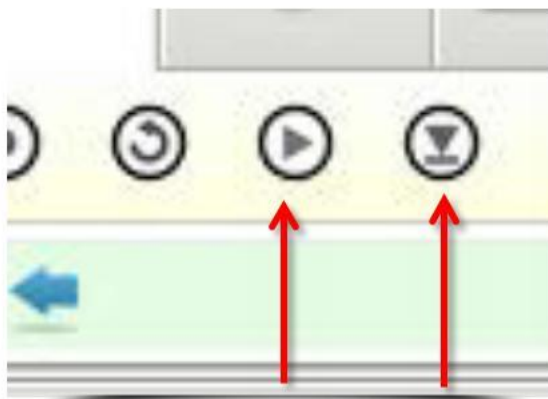

Monitoring Section
Extraction

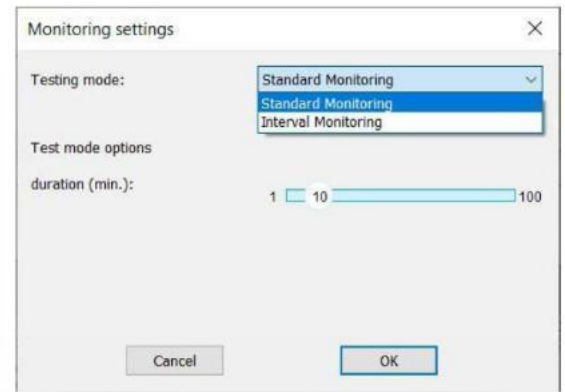

In addition to a time-independent measurement, you can also define your own run protocol for the monitoring function.

8

You can extract and separately save any sections of the measurement sequence for a later evaluation or analysis during the ongoing long-term measurement of the 'Monitoring' test mode.

All extracted sections made during the long-term measurement are numbered and highlighted in blue above the ECG display following the measurement.



For further analysis, each section extracted can be reviewed in isolation or in selective combination with other sections.




Example 'Extracted Section'
in blue = enabled

Example 'Extracted Section'
in yellow = disabled

Complex biological processes made fully transparent

A unique combination of reliable measurement methods

Traditionally, two main methods are distinguished in HRV analysis: the frequency domain method and the time domain method. While HRV systems often use only one of these methods for data recording, Nilas MV® uses both in combination.

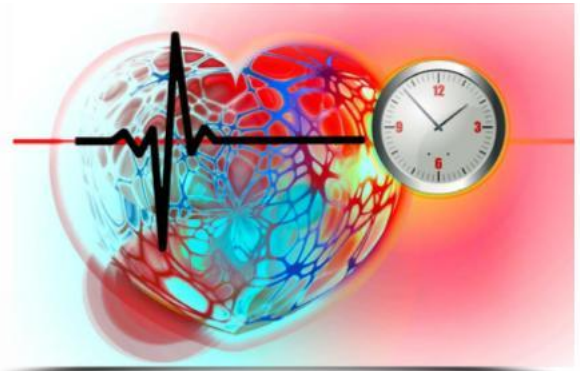


In the frequency domain the periodically recurring oscillations of the HRV with their different amplitudes are determined. These can be referred to physiological processes underlying HRV, the analysis of which allows a differentiated assessment of sympathetic and parasympathetic influences on the sinus node.


For analyses in the time domain, mean values and standard deviations of the R-R intervals are calculated and the HRV are displayed using descriptive statistical methods. These parameters provide specific information on overall variability as an expression of regulatory capacity and health (vitality).

In contrast to other measuring systems, Nilas MV® not only measures the actual R-R intervals, but also breaks down the HRV overall signal into its frequency components using spectral-analytical methods. This makes it possible to record the variance of the individual types of intervals and to relate them to each other in order to depict the events in the entire organism in an even more differentiated manner and to make them accessible to the user.

9

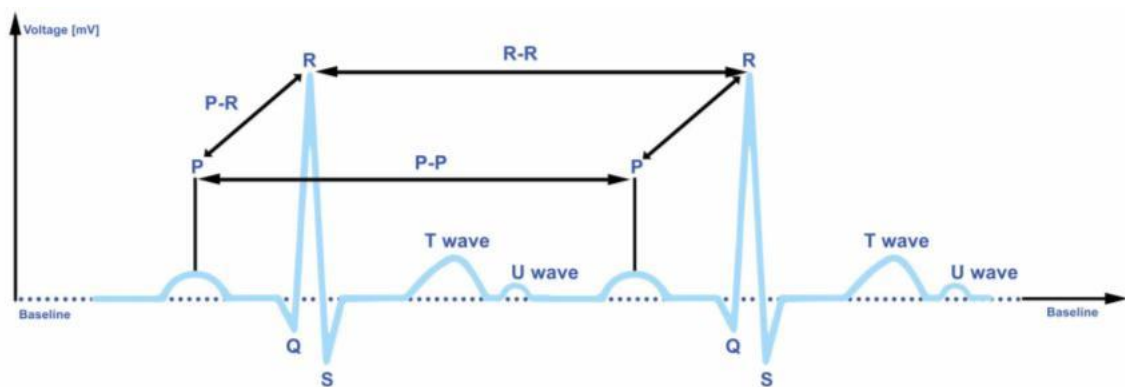

Nilas MV®: Measurement of real frequency components

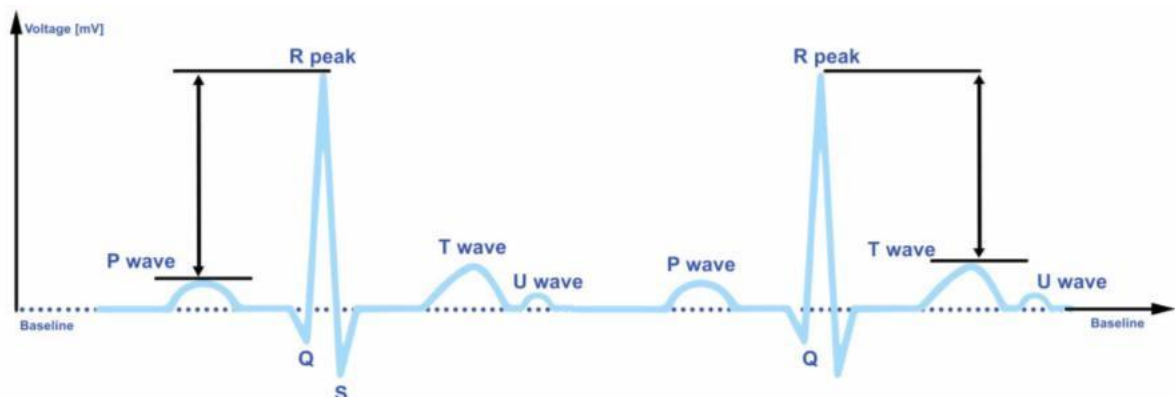


Nilas MV®: Measurement of real intensity parameters

Versatile functions for multi-purpose needs

The measurement and evaluation functions of Nilas MV® cover the areas of diagnostics, treatment, control and prognosis in a simple and efficient way.

Right on the heartbeat - The Rhythmogram


With the Rhythmogram the heart rate variability as well as sympathetic and parasympathetic influences on the sinus node are recorded and depicted on the basis of the heartbeat intervals.

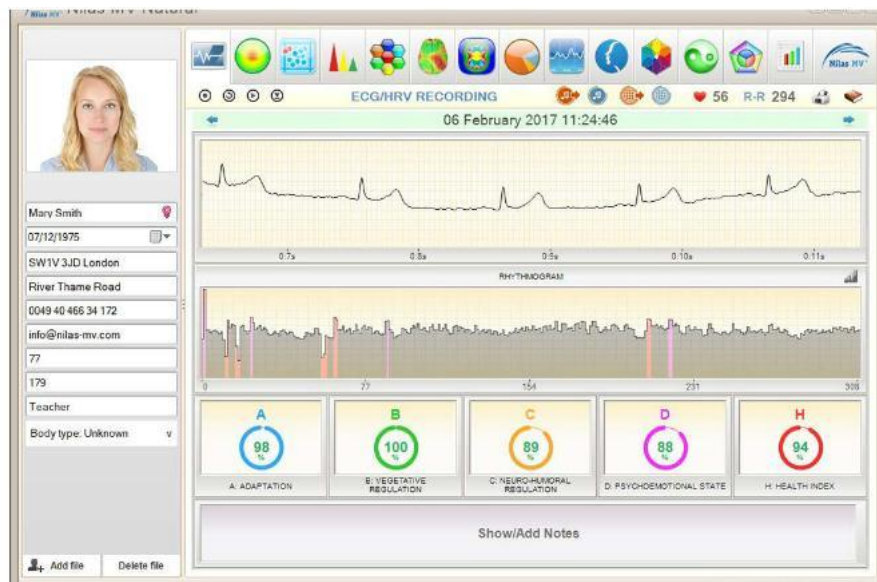

In contrast to a pulse measurement procedure, the real-time recording of Nilas MV® measures the exact peak of the heartbeat. Thus, Nilas MV® delivers measurement results in an accuracy margin of one millisecond with a yet unattainable degree of precision in HRV recording by capturing the exact length and intensity of every single heartbeat.

In addition, the software automatically removes all artifacts from the evaluation and thus enables a time-saving specific evaluation.

The dynamics of health

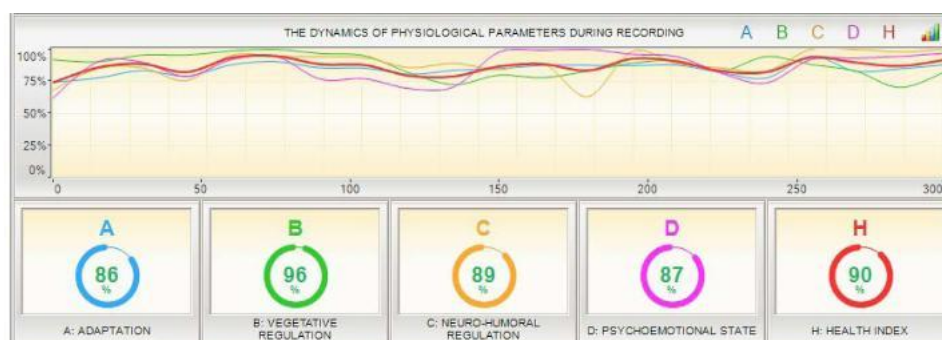

The dynamic function provides global information on the overall variability. The curves display the energy level and the resource consumption of physiological and psycho-emotional adaptation and regulatory parameters as well as their harmoniousness and interactions. This allows for concrete conclusions in the assessment of causes with deficits and increased need for regulation.



The autonomic nervous system - unconscious nervous system

Health and well-being are directly dependent on the proper functioning of the autonomic nervous system. In healthiness the influences of the sympathetic and parasympathetic nervous systems alternate with one another without falling into extremes. Shifts in this health balance, e. g. due to stress and in particular persisting stress, can be identified and analysed with Nilas MV®.


The frequency spectrum diagram shows the percentage activity level in the distribution of the three frequency segments:

- parasympathetic nervous system
- sympathetic nervous system
- neurohormonal system




The index of stress evaluation informs to what extent the overall constellation of the frequency spectrum is in balance.

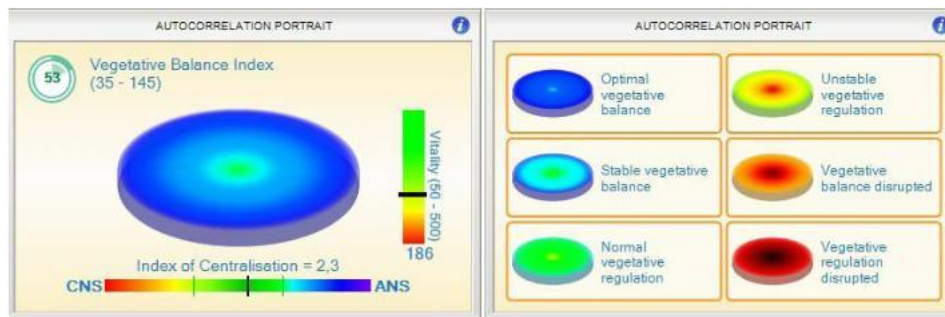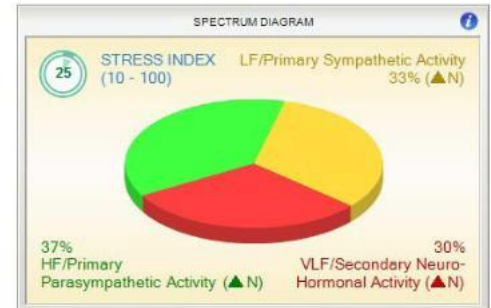


The autocorrelation portrait provides information on the current capability to adapt to stress. The clear legend enables a simple and quick classification of the measurement results.

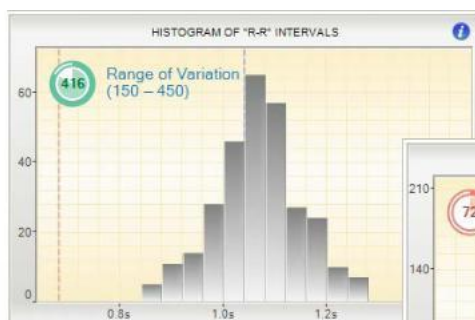

The histogram depicts the frequency of different RR intervals. The number of bars indicates the width of the heart rate variability.

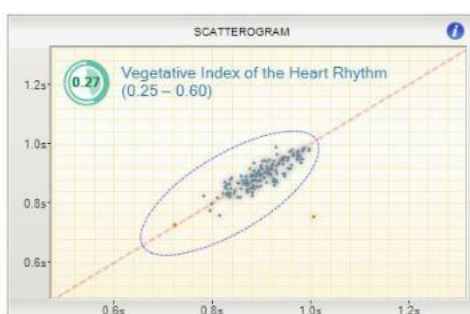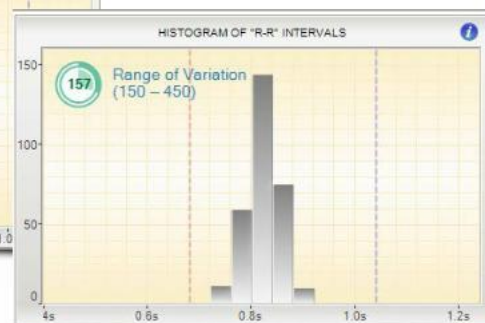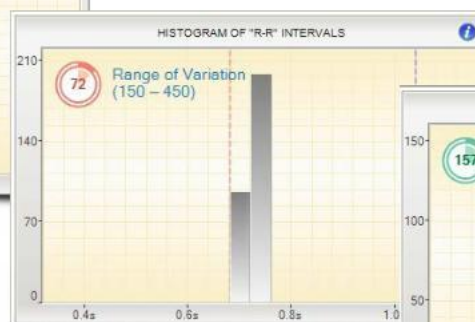

The Scatterogram is a two-dimensional depiction of the heart rhythm and a summary of the entire RR time series of the electrocardiogram.

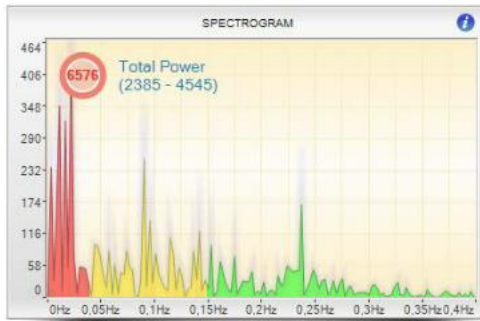


The stress level is scaled in the frequency spectrum. It provides information to you about the intensity and distribution of the individual signal components: Very low frequency (neurohormonal part), low frequency (sympathetic part) and high frequency (parasympathetic part). The "Total Power" index shows the measured total energy of the organism in the frequency spectrum (HF, LF, VLF) and its distribution as a numerical value.


In the risk analysis you will find a quick overview of important parameter values of the vegetative regulation. In addition to relevant HRV core values, it also includes indices for systemic processes.



Central Regulation

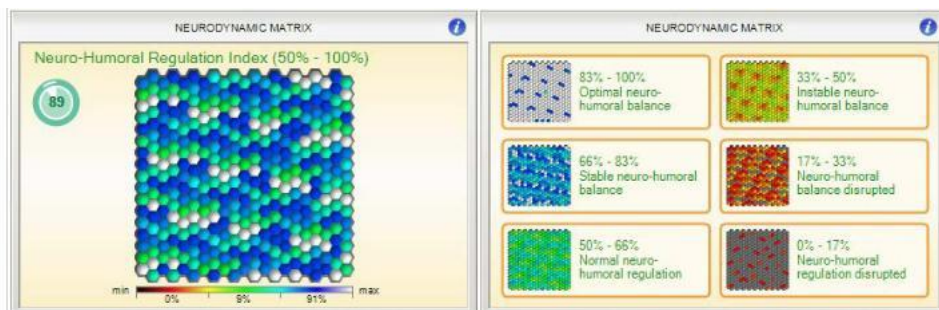

The periodically recurring oscillations of HRV with their different frequencies can mainly be assigned to individual physiological processes underlying HRV. The Neuro-Dynamic Matrix evaluates the balance and dynamics of physiological adjustment processes between the central nervous system and internal organismic subsystems.

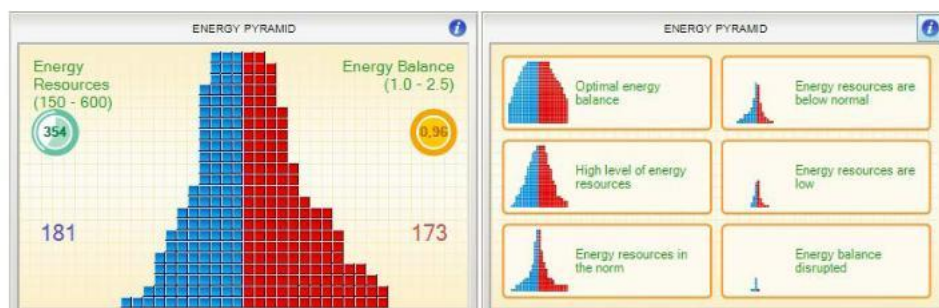


The pyramid shows the energetic equilibrium in the different organic and subsystems with the aid of blue (anabolic = building up) and red (catabolic = degradation) elements and values. It compares the energy intake and energy consumption of the organism.

Psycho-emotional state




The spline map is a topographic matrix simulation of the active functional areas of the central nervous system. Nilas MV® measures the condition of each heartbeat at intervals of 50 beats and displays it in its entirety as a two-dimensional animation.

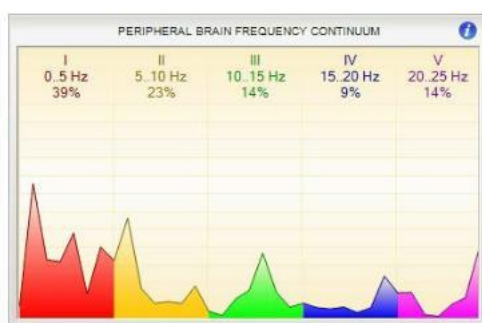

The neuroelectric activities during the measurement period are closely related to the psycho-emotional state and do range within certain frequency spectra, which are recorded with Nilas MV®.


The harmony of biorhythms

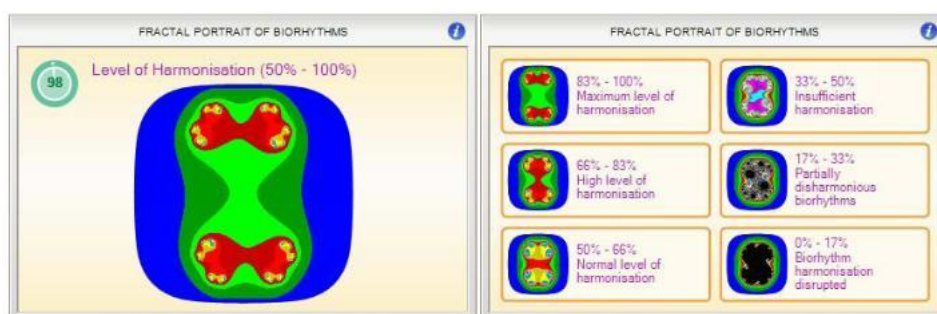

Through the fractal analysis of the biorhythms, Nilas MV® provides information on the long-term consequences of stress load and regulatory loss. It illustrates to what extent the different biorhythms of the organs and subsystems of the organism harmonise with each other.


The biological age

Vitality means that the biological age is younger than the calendrical age. Aging processes are intensified if the organism does not have sufficient opportunities for regeneration and compensation. The gerontological curve from the HRV measurement with Nilas MV® allows for conclusions to be drawn about the biological age of the body.

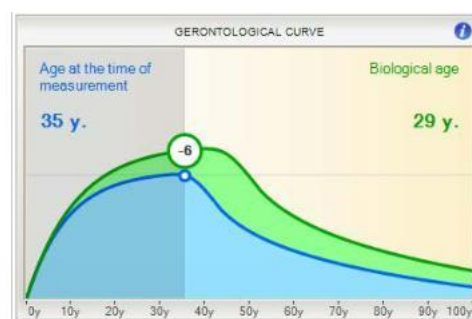


Everything at a glance: The overall assessment

The global analysis provides an immediate overview of the ratio and balance of the regulatory effective functional areas among each other and thus, where applicable, of deficits in the overall functional state.

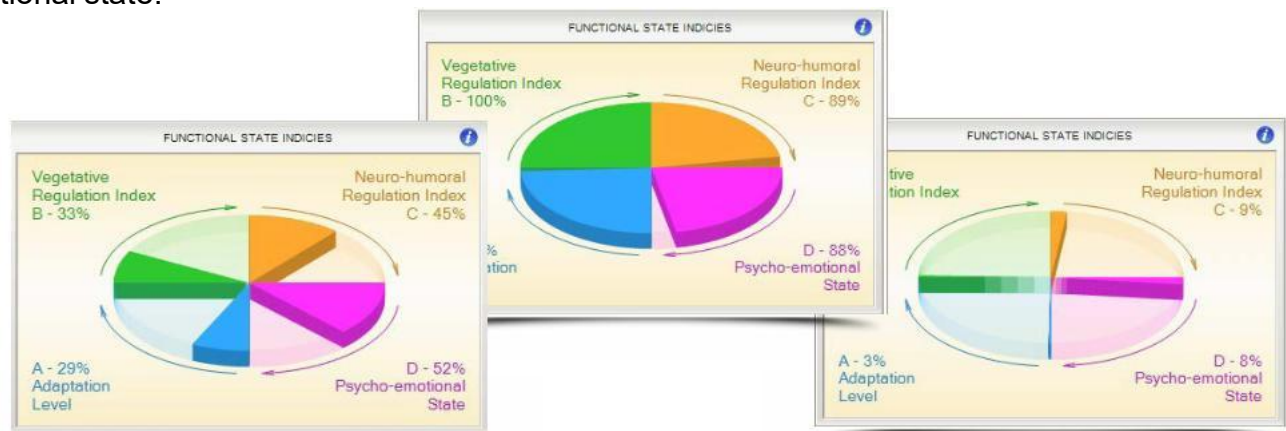

The age-related health index shows the organism's capability to regulate itself. The thin blue disc always represents the statistical average (target value), while the other-coloured circle segments represent the actual condition (actual value). The calculated value in % shows to what extent the individual health level is above or below the age-related statistics.

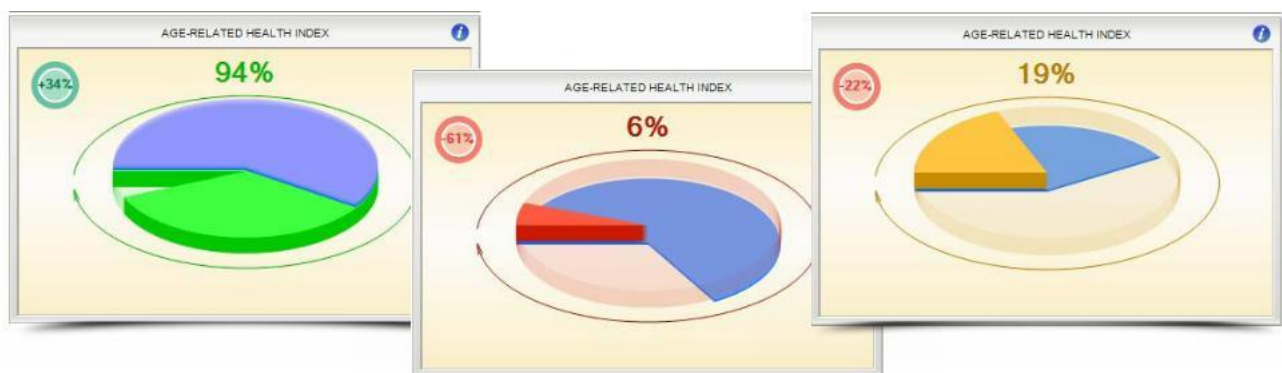

Progress monitoring




This overview shows the development of the indices of the functional state from all measurements of the same person tested. The dynamics of the values of the functional state are therefore qualified for follow-ups. The Circadian Forecast of Performance Capability shows the individual output curve over a 24-hour period.

Bio-Energetic Field



This depiction visualises the available total energy and its distribution in the different body regions. It is used to assess the integrity of the bio-energetic field. It projects the quality of energetic control circuits via shape character and shape density.

Chakras Activity Map

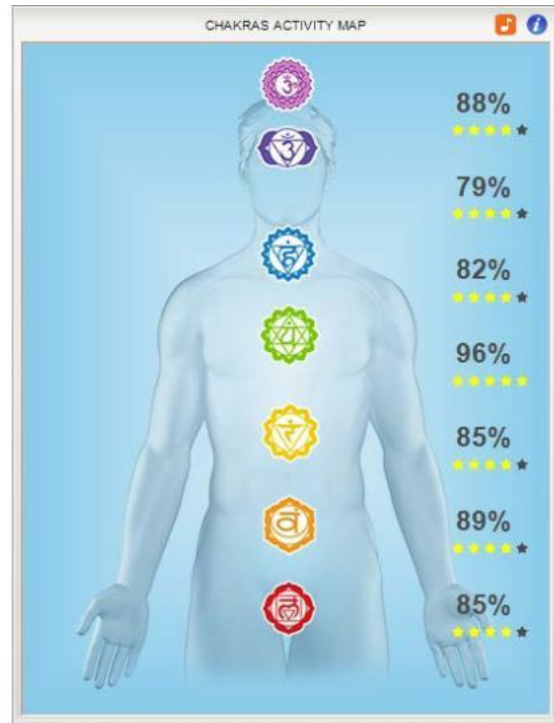

In addition to the current snapshot of a measurement the biological data is allocated to the Far Eastern chakra system with the help of specific algorithms allowing to explore general personality-associated characteristics.

The Chart of the Meridians



The Chart of Meridian includes different analysis and evaluation parameters derived from the 12 organ system meridians of the Traditional Chinese Medicine (TCM). Both the meridian diagram and the organ body clock can be used for diagnostic and therapeutic purposes.

Ayurveda Test: The Doshas and the 5 Elements

A system-integrated Dosha test can be used to determine the individual Dosha type, the individual ratios of the dosha elements as well as the Ayurvedic nutritional category.

Body Part/Tissue structure	Category 1	Category 2	Category 3
Body shape + height	slender physique; tall or very small stature	normal physique, medium-size growth	sturdy physique or large, strong growth
Body weight	tends to underweight	normal weight	tends to obesity
Shoulders	slender	normal	broad
Wrists	slender	normal	broad



Constitution-related Dosha Diet

In addition to certain characteristics and preferences, each constitutional type also has a tendency to specific imbalances in well-being. According to Ayurvedic teachings, the selection, combination and preparation of food can have a harmonising effect on the affected Dosha components.

Vata Constitution (Diet)

In a Vata-dominated constitution, the metabolism is active and dynamic. The digestive fire (Agni) is usually fluctuating and restless and the appetite is irregular. Sometimes Agni is strong (good absorption) but the digestive performance is significantly weakened. An increased Vata is accompanied by meteorism and changeable bowel movements.

Fundamentals for Vata harmonisation:

- Regular food intake
- At least 3 warm meals daily (ensure sufficient supply of good fat and protein!)
- Bigger dinner, fast, don't eat


Overview of Parameters

OVERVIEW OF PARAMETERS			OVERVIEW OF PARAMETERS		
Parameters	Values	Norms	Reference		
Parameters Heart Activity			Parameters Heart Activity		
HR, (beats/min)	88	[60-90]	Pulse		
IAB	96.6	[35.0-145.0]	Differences RR-Intervals		
VIHR	0.46	[0.25-0.60]	Vegetative Index of Heart Rhythm		
IAR	44.1	[15.0-60.0]	Index of Appropriate Regulation		
SI	80.5	[10.0-100.0]	Stress Index		
Parameters Vegetative Regulation			Parameters Vegetative Regulation		
B1 (%)	82	[60-100]	Regulatory Level		
B2 (%)	94	[60-100]	Regulatory Reserve		
Statistical Parameters			Statistical Parameters		
RRNN (ms)	680	[700-1000]	Duration of the Middle Interval		
SDNN (ms)	63.4	[30.0-100.0]	Standard Deviation of Middle Interval results		
CV (%)	9.3	[3.0-12.0]	Variation coefficient		
RMSSD (ms)	32.7	[15.0-45.0]	Root-mean square differences of successive R-R intervals		
NN50	39	[46-100]	Number of Spontaneous Changes in Intervals of Longer than 50 ms		
pNN50 (%)	13	[15-34]	Percentage of Spontaneous Changes in Intervals of Longer than 50 ms		
Frequency Spectrum			Frequency Spectrum		
HF (ms²)	360	[770-1078]	High Frequency Power (0.15 Hz — 0.4 Hz)		
LF (ms²)	1538	[754-1586]	Low Frequency Power (0.04 Hz — 0.15 Hz)		
VLF (ms²)	1923	[600-1500]	Very Low Frequency Power (0.00 Hz — 0.04 Hz)		
HFnu	18.96	[26.00-32.00]	HF Frequency Performance (0.15 Hz — 0.4 Hz) in standardised units		
LFnu	81.04	[50.00-58.00]	LF Frequency Performance (0.04 Hz — 0.15 Hz) in standardised units		
LF/HF	4.27	[1.50-2.00]	Ratio of Vegetative Balance: sympathetic(LF)/parasympathetic activity(HF)		
TP (ms²*1000)	3821	[2385-4545]	Total Power of the Entire Frequency Spectrum		
Parameters Histogram			Parameters Histogram		
Mo (ms)	600	[700-900]	Mode		
AMo (%)	26.46	[30.00-50.00]	Amplitude of the Mode		
VR (ms)	274	[150-450]	Range of Variation		
HRV Index	16.0	[20.0-50.0]	Heart Rate Variability Index		
Scatterogram			Scatterogram		
SD1	23.1	[10.0-40.0]	Short-term HRV changes		
SD2	89.6	[50.0-121.0]	long-term HRV changes		
SD2/SD1	3.9	[1.2-4.0]	ratio of long- and short-term HRV changes		
Parameters Autocorrelation			Parameters Autocorrelation		
1k	0.864	[0.700-1.000]	Correlation Coefficient After the First Shift		
m0	31	[0-18]	Number of Fluctuations With a Value of Less Than 0		
Z	311.7	[50.0-500.0]	Autocorrelation Index		
IC	1.0	[1.0-3.0]	Index of Centralisation		


The raw data of the cardiac activity parameters are listed in this overview. In addition to the measuring ranges for HRV, it contains their standard and guideline values, the individually measured results and a legend with the associated units.

The power of breath for health and fitness - The Breathing Module of Nilas MV®

Breathing is an essential life function primarily intended to supply oxygen and expel carbon dioxide. However, how we breathe is also very important and can enable us to remain healthy, rectify disorders, improve energy supply and balance our physical and mental well-being. Nilas MV® creates based on the measured heart rate variability an individual respiratory rate pattern designed for a 5 minute breathing exercise.

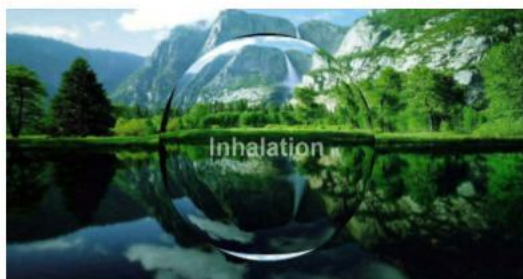

When done according to the instructions, it helps to restore our own ideal rhythm, simultaneously regulating biorhythms, strengthening the parasympathetic system, reducing stress and increasing our energy levels as well as psychic abilities like concentration. It also serves for testing the individual regulatory capacity on the basis of the body's resources.

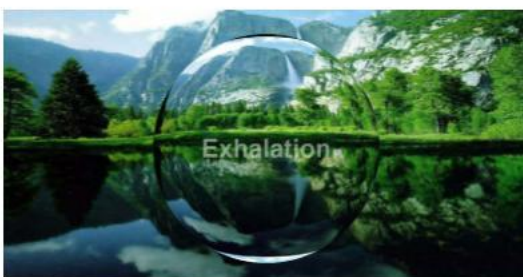

The respiration procedure is illustrated with the aid of a dynamic ball, which expands to correspond with when we breathe in and contracts as we breathe out. The pace of the expansion and the contraction of the ball as well as the length of the breather are determined by the results of the measurement data and individually tailored to the subject of each test.

The relaxation effect is enhanced by the interplay between the breathing ball, varying landscape scenery and embedded classical music.

The breathing module can be directly saved as a video file, burned on CD, saved on a USB stick, or delivered to the user via a file-hosting service.



Getting back to balance by setting the right tone - The Sound Module of Nilas MV®

Long ago, our ancestors realised the profound impact that the property of matter we call SOUND, has upon our lives. Recent research, carried out by Dr. Luciano Bernardi (Pavia University, Italy), shows that the heart responds to match the rhythm of the music being listened to.

The Nilas MV® software makes it possible to generate for each person a unique audio file containing a sound pattern designed to correct bio-energetic imbalances and deficiencies. It uses the difference between the optimal state and the actual state of the neuro-dynamic matrix to identify which signals are needed by you, to restore the body's optimal state of balance, which it then superimposes onto a background of natural sounds.



every day for a period of two weeks. When after two weeks, the neuro-dynamic matrix adapts to the course of the sound regulation, it makes sense to conduct another scan to generate a new sound pattern file which calibrates to maintain and further improve the new state of health and well-being.

The file is approximately 20 minutes in duration and should be listened to



Scope of Delivery

Scope of Delivery	
Position	Quantity
Hardware unit for HRV recording (cardiograph)	1
Software, inkl. monitoring feature, report feature for documentation, comment function, regulatory modules	1
Reusable peripheral clamp electrodes	2
USB 2.0 connection cable type A/B	1
Security dongle	1
Service	
Warranty/Technical Support	2 years included
Online training and special consultation	upon request
Updates	available separately
System requirements for computers**	
Operating System Windows: Windows 7 or higher	
Processor: Intel i5 or higher	
RAM: min. 4 GB	
Hard Disc min. 32 GB	
**Nilas MV® is a Windows-based software	

Space for your notes



Devices for holistic Health and Medicine



VitaCellMedical GmbH

Am Sonnenhang 44

D-67714 Waldfischbach-Burgalben

Tel.

+49 6333 9909030

E-Mail

info@vitacellmedical.com

Internet

www.vitacellmedical.com